# ERGO-LINE

**ERGO-STYLE / ERGO-LUX / ERGO-BASE**

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**Instructions for use**

**Mode d'emploi**

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## **ERGO-Style, ERGO-Lux und ERGO-Base**

You have chosen a magnifying glass of the ERGO-Line series, a top quality SCHWEIZER product manufactured in compliance with currently valid norms and regulations. A branded product Made in Germany. Congratulations on your decision.

No matter which model you have chosen, the products of the **ERGO-Line** series are a fusion of design and function, a symbiosis of lifestyle and high utility value.

### **Tips for usage**

If you use the **ERGO-Style** or the illuminated **ERGO-Lux** on a table, you can rest the end of the handle at the table top (illustration 1). Thereby you will have a tranquil and stabilized support. If then the lens is parallel to the text the best reading distance is found. The magnifying head **ERGO-Base** completes

the illuminated **ERGO-Lux** to a complete illuminated stand magnifier. Here you can adjust the tilting to a comfortable posture easily. You can separate the magnifier from the base by the magnetic connection (patent pending) anytime.

Please read these instructions carefully before using your **ERGO-Line** magnifier and always observe the care and safety instructions.

### **General safety notes**

**Fire hazard !!!** Never leave your magnifying glass lying uncovered in the sun. The intensifying effect of the lens can start a fire.

### **Danger of injury!!!**

- Never use your magnifying glass looking in the sunlight.
- You must not look directly into the LED, as otherwise there is the risk of an eye injury.

- Avoid mechanical damage such as knocking, scratching or dropping it, avoid moisture and high temperatures.
- Please note:  
The magnetic connection between magnifier and base is strong enough to common usage. But whilst carrying around a part could fall down an be damaged.

**Used batteries should not be disposed of** with normal domestic waste, please ensure that they are **properly disposed of**. Inform other persons, especially children, of these hazards.

### **General notes**

Switch on the light by pushing the switch on the handle forward in the direction of the lens; push back to switch off (illustration 2).

### **Changing the batteries** (illustration 3 and 4)

Switch off the light. Open the lift-up battery compartment at the end of the handle and replace the old batteries with new ones. Slide three AAA batteries with their plane end first into the battery compartment. Additionally you will find a + marking for the correct polarity on the battery compartment. For closing the compartment, snap shut the cap. Always use alkali manganese batteries (1,5 V) to ensure extremely long life with no loss of lighting quality.

### **Using the magnifying heads ERGO-Base** (illustration 5 and 6)

You can connect the illuminated magnifier **ERGO-Lux** to the magnifying head **ERGO-Base** via the magnetic connection. Position the magnifier that way to the base that the handle is oriented towards the cut-out of the magnifying head. Now you can incline the magnifier to a comfortable position. To separate

the the magnifier from the head simply hold the head and lift the magnifier upwards.

## **Caring for your illuminated magnifying glass**

Clean the lens with a damp cloth (a damp microfibre cloth is ideal). Never use solvents such as alcohol or white spirit or household cleaners or washing up liquid as these could damage the lens.

## **After-sales service**

Should your magnifying glass of the series **ERGO-Line** show a defect please take it to the retail location where it was bought for return to SCHWEIZER for repair.

## Versions ERGO-Line

Lens Size	Lens strength	Magnification D/4	Distance eye - lens	image distance
55 mm Ø	24 D	6 x	100mm	400mm
55 mm Ø	20 D	5 x	120mm	400mm
60 mm Ø	16 D	4 x	140mm	400mm
70 mm Ø	12 D	3 x	170mm	400mm
100x75 mm	8 D	2 x	190mm	400mm

## Technical notes

Three normal alkali manganese batteries (micro 1,5 V, LR 03, AAA, AM4, MN2400) are required as the power supply.

The special LED has a life of around 10.000 hours of constant use and never has to be replaced due to its high durability.

### Guarantee:

We provide a guarantee of **ten years** following the date of purchase (**as shown on receipt**) for the full function of the LED illuminated magnifying glass and for any defects occurring within the guarantee period which are attributable to material or manufacturing errors.

Damage caused by incorrect use or due to impact is not covered by the guarantee. Also the batteries are not covered by the guarantee.

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A. Schweizer GmbH