


# TV Magnifier Mounting Instruction

## CTT-37W & CTT-41W

1. Put four props **1** into two tube **2**
2. Put **3** caster in Tube **2**
3. To link **4** and **5** Tubes by screws, Assemble to join **2** full Base Finished, become standing on the ground.
4. Use Tube **6** to join Tube **2** by screw to make them become  $\perp$  bracket.
5. Adjust tubes **6** **7** up or down to operate in the height of your TV.
6. Move full set Magnifier to the front of TV sets, Open TV set and adjust TV tube toward the center of Magnifier. If not, Tube **8** & **7** until getting a properly.

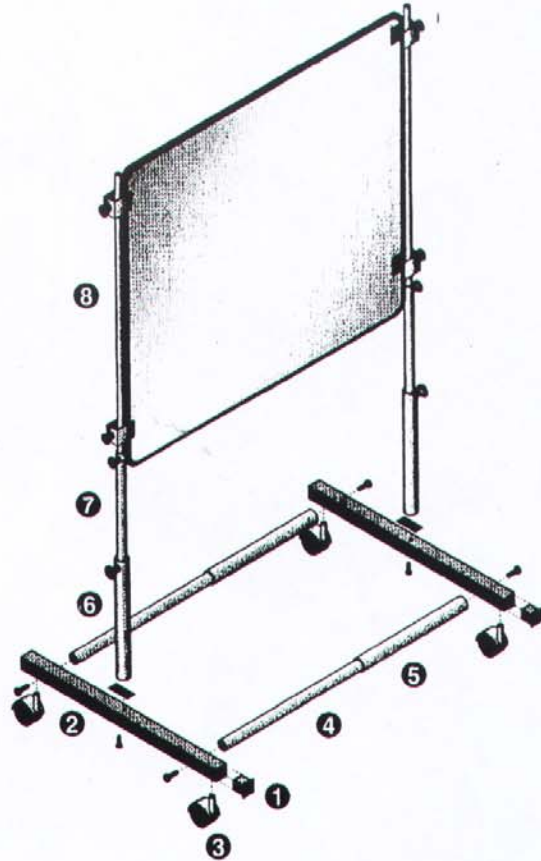

### \*Sand Bracket Part:

1. Caster 3 by 4
2. Tube 8 by 2
3. Props 1 by 4
4. Tube 7 by 2, 6 by 2    Length : 32cm,  
4 By 2 Length: 55cm, 5 by 2 : 30cm

**Important:** When handing TV-Magnifier be careful not to damage Surface of the Lens.

**Warning:** Do not leave lens in direct sunlight, since it will Act as a Focal lens and may cause burning.