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GB Congratulations!

You have selected a magnifier from SCHWEIZER, a quality product manufactured according to the currently valid standards and regulations. A "Made in Germany" brand. Congratulations on your purchase.

Please read these instructions for use carefully before using your magnifier for the first time and observe the safety, cleaning and care information.

GB General information

The lens in your SCHWEIZER magnifier is made from glass or special plastic. The optical quality of both materials is second to none. Lenses made from optical glass are ground and optically polished on both sides. The hard surface features a high chemical and mechanical resistance. Only UV-absorbing crown glass, which is also used for the production of prescription lenses, is used in the manufacturing process.

Lightweight plastic lenses are made from optically pure Plexiglas®. Computer-optimised aspheric surfaces provide clear edge-to-edge images and ensure optimal use of the entire field of view.

Cleaning and care

With proper care, your magnifier will provide you with many years of untroubled reading.

Clean the lens with a damp, lint-free cloth (ideally a damp microfibre cloth). Never use solvents such as alcohol, benzene or other cleaning agents for cleaning lightweight plastic lenses since they can damage the lens. You may find small lines that look like hairline cracks at the edge of lightweight lenses or in the plastic frame. These are flow lines that occur during the production process; they do not affect the function or durability in any way.

Safety instructions

Fire hazard! Never leave your magnifier out in the sun uncovered. The intensifying effect of the lens can start a fire.

Risk of injury!

- Never use the magnifier to look at the sun.
- Do not look directly at the light source of illuminated magnifiers.
- Avoid mechanical impact or shock such as banging or dropping your magnifier, and ensure it is not exposed to moisture or to high temperatures.

Inform other people about these risks, especially children. Do not dispose of used batteries along with normal refuse. Please ensure proper disposal (see page 13).

Power supply

The mains-operated version of the TWIN-Lux LED requires mains voltage. The ERGO-Lux i mobil series uses a built-in lithium-polymer battery. The ÖKOLUX plus mobil and ERGO-Lux MP series use three standard AAA alkaline batteries (1.5 V, LR3, AM4). The ÖKOLUX plus and MODULAR LED series as well as the battery-operated TWIN-Lux LED use three standard AA alkaline batteries (1.5 V, LR6, AM3).

The LED light-emitting diodes are subject to the most stringent selection criteria. They have a lifespan of more than 50,000 hours with continuous operation. Replacement of the LED is thus not normally required.

GB Battery safety instructions

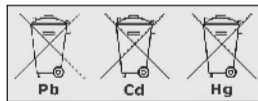
We place our batteries on the market in line with legal requirements. Please read the points below on the safe disposal of our batteries:

- To prevent harmful effects to the environment and health, used batteries must be disposed of properly at recycling centres or returned to the point of sale. Returning batteries is required by law and free of charge for you.
- Batteries should only be returned in a fully discharged state. The poles of lithium-ion batteries must be insulated with adhesive tape.
- Valuable raw materials such as zinc, iron, nickel and so on can be recovered by recycling used batteries.
- The batteries can be easily removed from our devices

for disposal (see the following pages). Magnifiers containing a built-in rechargeable battery must be disposed of completely.

- The “crossed out wheelie bin” symbol indicates that it is illegal to dispose of used batteries along with normal refuse.

The additional specification of the chemical symbols for lead (Pb), cadmium (Cd) and mercury (Hg) indicates that these materials are contained in the batteries in legally permissible quantities.

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GB Also available from SCHWEIZER:

- Europe's largest range of special visual aids
 - Special contrast-enhancing spectacle lenses
 - Special magnifying lenses
 - High addition spectacles
 - Telescopic spectacles
 - Low vision lighting and much more
- Video magnifiers
- Magnifiers for technical applications
 - Precision folding magnifiers
 - Microscope graticules
 - Linen testers
 - Watchmakers eyeglasses
 - Table top magnifiers, illuminated

18-001/120188/07

SCHWEIZER

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Copyright © April 2018 A. SCHWEIZER GmbH
- ⒼⒷ All rights reserved.