# Instructions for use

## for the inductive magnifier ERGO-Lux *i* mobil

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Congratulations on purchasing the ERGO-Lux *i mobil* magnifier. You have selected a magnifier from SCHWEIZER, a quality product manufactured according to currently valid standards and regulations. A brand made in Germany.

Your new magnifier uses a built-in lithium-polymer battery that is charged by induction.

The information and data contained in this document are subject to change without notice. No part of this document may be reproduced or transmitted in any form or by any means, electronic or mechanical, or for any purpose, without the express written approval of A. SCHWEIZER GmbH.

Copyright © March 2017 A. SCHWEIZER GmbH. All rights reserved.

# Table of contents

GB

<b>Please read before first use</b>	<b>6</b>
General information	6
Cleaning and care	6
<b>General safety instructions</b>	<b>7</b>
<b>Battery safety instructions</b>	<b>8</b>
<b>Charging the ERGO-Lux i mobil with the charging pad</b>	<b>9</b>
Connecting the charging pad	9
Charging the inductive magnifier	9
Handling and storing the inductive magnifier	10
Qi standard	10
<b>Using the ERGO-Lux i mobil</b>	<b>11</b>
Switching on/off	11
Selecting the brightness level	11
Attaching a lanyard (neck/wrist strap)	12

RoHS

# Please read before first use

Please read these instructions carefully before using the magnifier for the first time and observe our safety, cleaning and care information.

GB

## General information

Your magnifier contains a lightweight plastic lens made by SCHWEIZER from optically pure Plexiglas®. Computer-optimised aspheric surfaces provide clear edge-to-edge images and ensure optimal use of the entire field of view.

## Cleaning and care

Proper care of your magnifier means it will provide you with many years of untroubled reading. Clean the lens with a damp, lint-free cloth (ideally a damp microfibre cloth). Never use solvents such as alcohol, benzene or other cleaning agents since they can damage the lightweight (plastic) lens. You may find small lines that look like hairline cracks at the edge of lightweight lenses or in the plastic frame. These are flow lines that occur during the production process; they do not affect the function or durability in any way.

# Safety instructions

Fire hazard! Never leave your magnifier out in the sun uncovered. The intensifying effect of the lens can start a fire.

Risk of injury!

- Never use the magnifier to look at the sun.
- Do not look directly at the light source of illuminated magnifiers.
- Avoid mechanical impact or shock such as banging or dropping your magnifier, and ensure it is not exposed to moisture or to high temperatures.

Inform other people about these risks, especially children.

Do not dispose of used batteries along with normal refuse. For information on the proper disposal of batteries, please read the instructions on the next page.

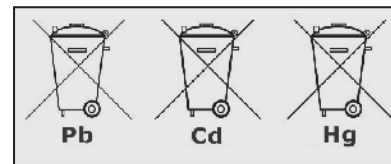
# Battery safety instructions

Your new magnifier uses a built-in lithium-polymer battery that is charged by induction.

We place our batteries on the market in line with legal requirements. Please read the points below on the safe disposal of our batteries:

- To prevent harmful effects to the environment and health, used batteries must be disposed of properly at recycling centres or returned to the point of sale. Returning batteries is required by law and free of charge for you.
- Valuable raw materials such as zinc, iron, nickel and so on can be recovered by recycling used batteries.
- Since the magnifier contains a built-in battery, it has to be disposed of separately from ordinary household waste at the end of its life.
- The “crossed out wheeled bin” symbol indicates that disposal of used batteries along with normal refuse is illegal.

The additional specification of the chemical symbols for lead (Pb), cadmium (Cd) and mercury (Hg) indicates that these materials are contained in the batteries in legally permissible quantities.


WEEE reg. no.: DE 69691942

# Charging the ERGO-Lux i mobil with the charging pad

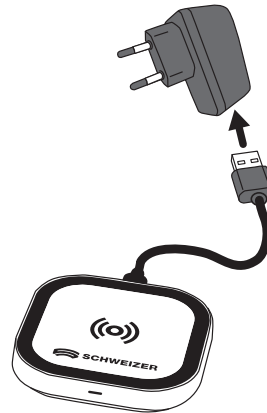
## Connecting the charging pad

Connect the charging pad and USB cable to a USB power adapter or any other suitable USB port. The indicator light at the front of the charging pad then turns green.

## Charging the inductive magnifier

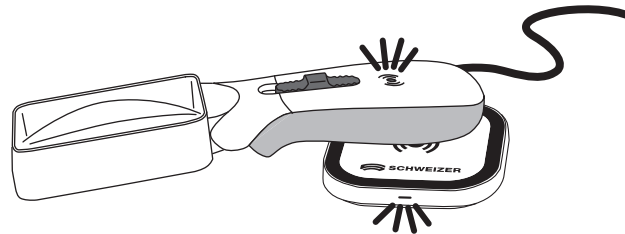To charge the magnifier, switch the device off and place its handle on the pad. The charging pad's indicator light will change to blue when the charging process starts. In addition, an indicator light on top of the magnifier will turn red during charging. This indicator light is very helpful when you are using a different charging pad with no indicator light of its own. If the indicator light on the magnifier surface does not turn red, reposition the handle on the pad.

Once the battery is fully powered, the indicator lights on the magnifier and charging pad will both switch off.

Note: You may hear a faint ticking sound while the battery is charging.  
If the magnifier's light is left on while charging, it takes longer for the battery to charge and the light may flicker.  
It takes approx. 3 hours to fully charge the ERGO-Lux i mobil.

## Handling and storing the inductive magnifier

To use the magnifier, simply remove it from the charging station. You can leave the magnifier on the charging station when it is not in use without any risk of overcharging. You can also leave the charging pad connected all the time.

Do not leave the battery in an uncharged state for a long period of time. If the magnifier will not be used for some time, it is best to store it charged.

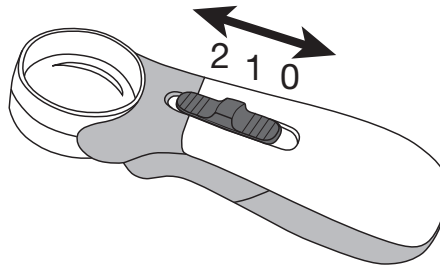
## Qi standard

Your magnifier uses the Qi standard for charging and is thus compatible with the majority of charging pads, e.g. for mobile phones.

# Using the ERGO-Lux i mobil

## Switching on/off

To switch on the magnifier, slide the grey switch up from the OFF position (0) to position 1 or 2.

## Selecting the brightness level

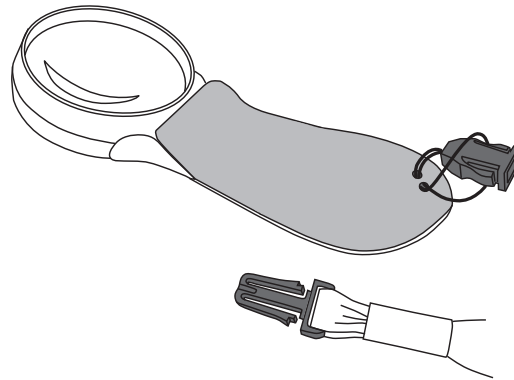
To adjust the light intensity of the magnifier while reading, you can select between 2 brightness levels:

Position 1 for regular illumination

Position 2 for extra-bright illumination (boost mode)

## Attaching a lanyard (neck/wrist strap)

ERGO-Lux i mobil features lanyard holes integrated into the handle for easy attachment of a neck or wrist strap. The strap makes it easy to keep the magnifier within reach at all times and prevents accidental loss. To attach the lanyard to the magnifier handle, turn the magnifier over and feed the small loop through the holes located on the lower handle end. Then thread the strap through the loop (see figure).

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Copyright © Mars 2017 A. SCHWEIZER GmbH. Tous droits réservés.
Copyright © Marzo 2017 A. SCHWEIZER GmbH. Todos los derechos reservados.
Copyright © Marzo 2017 A. SCHWEIZER GmbH. Tutti i diritti riservati.
Copyright © maart 2017 A. SCHWEIZER GmbH. Alle rechten voorbehouden.