**Konformitätsbestätigung / Declaration of Conformity / Déclaration de Conformité**

**Produkt / Product / Produit:**

CD0-006 Multilight LED Pocket

**Hersteller / Manufacturer/ Fabricant:**

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**(GB) CONGRATULATIONS!**

You've purchased the finest in natural lighting technology with your new SCHWEIZER product.

**BATTERY INSTALLATION/ REPLACEMENT**

Your Multilight LED Pocket is powered by 3 AAA batteries.

**Warning: Keep away from children. If swallowed, promptly see doctor. Do not install backwards, charge, put in fire or mix with other battery types – may explode or leak causing injury.**

- To install batteries, make sure cover (A) is closed and the light is facing down.
- Open the battery compartment cover (C) by sliding it away from the bottom of the light (Figure 1), then swing the cover open (Figure 2)
- Install the 3 AAA batteries as indicated by the graphic inside the battery compartment and Figure 3.
- Swing battery cover (C) down then push it toward the bottom of the light to engage tabs (D).
- To turn the light ON, flip open cover (A).
- To turn the light OFF, close the cover (A) until it locks.
- To attach e. g. to a belt, hold open clip (B) and slide the light over the belt. Or you can also fix a lanyard to the loop.

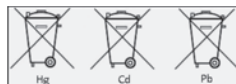
**Notice: Only replace batteries as a simultaneous set and do not mix with older batteries. Remove batteries if consumed or if the light will not be in use for a long period of time.**

Enjoy your Multilight LED Pocket!

**BATTERY SAFETY NOTES**

We would like to inform you that our batteries/rechargeable batteries are marketed in accordance with the current law and refer you to the following points:

- This product contains type AAA HR03 rechargeable batteries. You can also use type AAA LR03 batteries.
- To prevent harmful effects to the environment and health, used batteries/rechargeable batteries must be disposed of properly at community recycling centres or returned to the merchant. Returning batteries is required by law and is completely free-of-charge for you.
- Batteries/rechargeable batteries should only be returned in a discharged state.
- Valuable raw materials such as zinc, iron and nickel can be reclaimed by recycling used batteries/rechargeable batteries.
- The batteries/rechargeable batteries can be easily removed from our devices for disposal (see User's Guide).
- The symbol of a „crossed-out trash container“ indicates that it is illegal to dispose of used batteries with regular domestic waste.

The additional specification of the chemical sign for lead (Pb), cadmium (Cd) and mercury (Hg) indicates that these materials are contained in the batteries/rechargeable batteries in accordance with the legally-specified limits.

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**(GB)** Also Available in the MULTILIGHT LED Series:

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And here's what else you'll find at SCHWEIZER:  
Europe's largest range of special corrective  
lense

- Special spectacle lenses for increased  
contrast
  - Special spectacles with magnifying  
effect
  - Magnifying spectacles
  - Telescopic magnifying spectacles
  - LowVision illumination and much more
- Magnifiers for technical applications
- Precision fold-in magnifiers
  - Precision measuring magnifiers
  - Scale measuring magnifiers
  - Thread counters
  - Wand microscopes, illuminated
  - Watch makers' magnifiers
  - Work stand magnifiers, illuminated
- Electronic Aids

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