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Instructions for use of MULTILIGHT LED Lamps

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Ⓔ Congratulations!

You've purchased the finest in LED lighting technology with your new SCHWEIZER product.

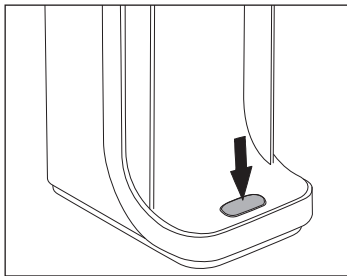
Please read these instructions for use completely before using your magnifier for the first time and observe the safety and care instructions.

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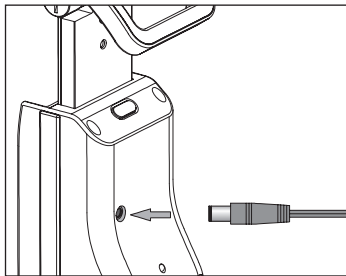
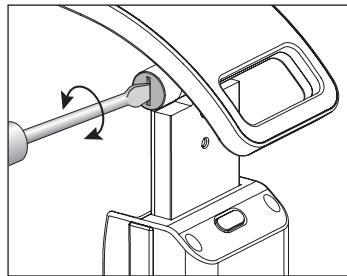
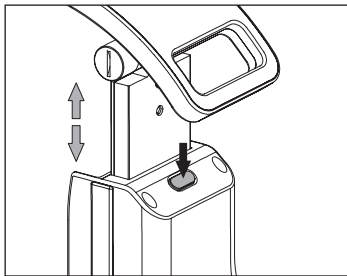
(GB) Reload empty battery approx. 12 hours. Burn time up to 3 hours. Replacement for depleted battery at your distributor.

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
OttLite Technologies, Inc.

1214 West Cass Street, Tampa, FL 33606

RICHTLINIEN / DIRECTIVES:	NORMEN / NORMES:
Low Voltage Directive 73/23/EEC	EN60598-2-4 Electrical Safety for Luminaries EN 62471:2008 Photobiological Safety
EMC Directive 2004/108/EC	EN55015 EMC for Luminaries
	EN61547 EMC Immunity
	EN61000-3-2 Current Harmonics

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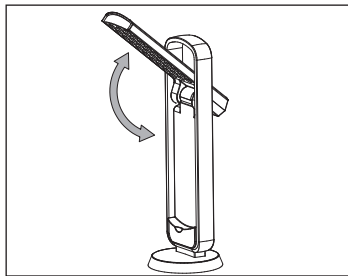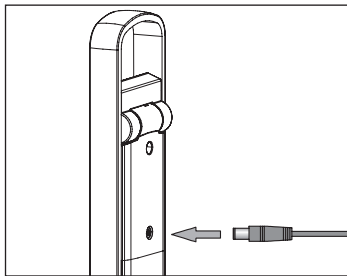
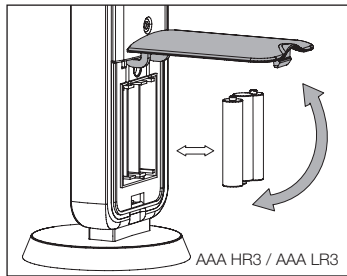
GB Using the A/C adapter automatically disconnects the batteries. The A/C adapter only provides auxiliary power, it will not charge batteries.

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
OttLite Technologies, Inc.


1214 West Cass Street, Tampa, FL 33606

RICHTLINIEN / DIRECTIVES:	NORMEN / NORMES:
Low Voltage Directive 2006/95/EC	EN60598-2-4 Electrical Safety for Luminaries EN 62471:2008 Photobiological Safety
EMC Directive 2004/108/EC	EN55015 EMC for Luminaries
	EN61547 EMC Immunity
	EN61000-3-2 Current Harmonics

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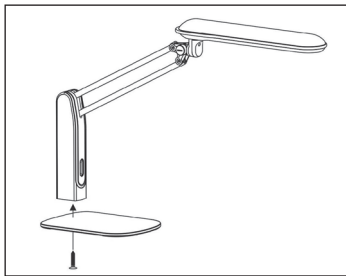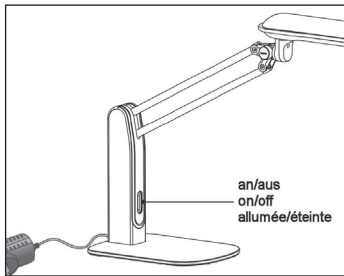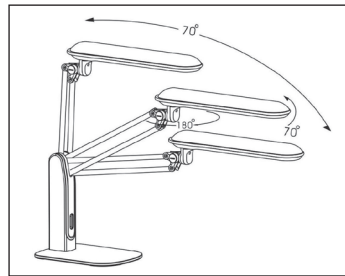
GB To screw together the base and the lamp, remove the screw from the base and use it to screw together the base and the lamp.

Insert the plug into the socket. Please make sure that the connection cable cannot be tripped over.

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MULTILIGHT LED Table


Hersteller / Manufacturer / Fabricant:

Zoom Electronics Ltd., No. 5 Meijing East Road
Gangkou Town, Zhongshan-Guangdong

RICHTLINIEN / DIRECTIVES:	NORMEN / NORMES:
Low Voltage Directive 2006/95/EC	EN60598-2-4 Electrical Safety for Luminaires EN 62471:2008 Photobiological Safety
EMC Directive 2004/108/EC	EN 55015 Luminaries EN 6100-3-2 Immunity EN 6100-3-3 Current Harmonics EN 61547 Limitation of voltage changes

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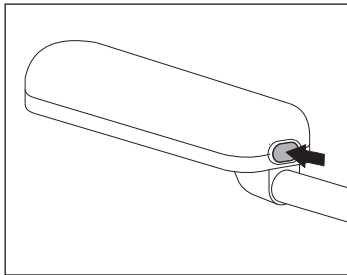
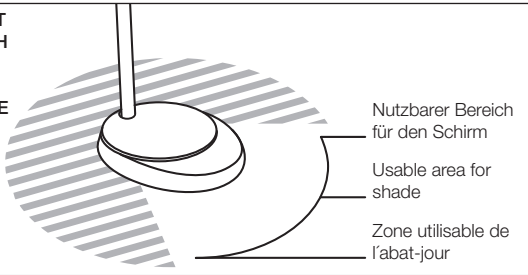
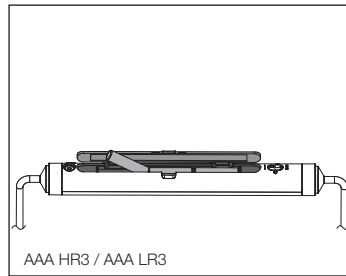
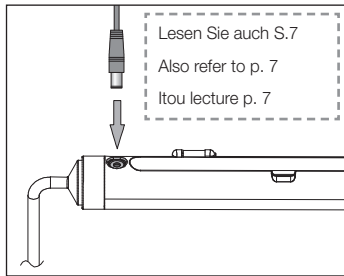
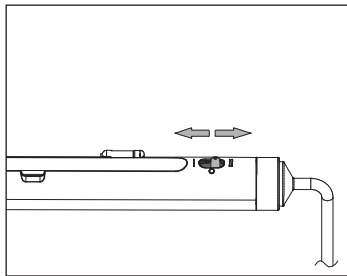
MULTILIGHT LED Leseput / Bookstand / Porte-livre

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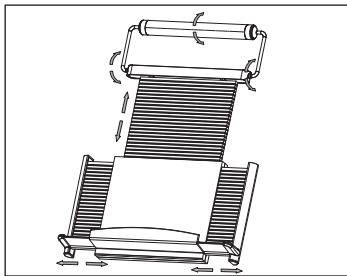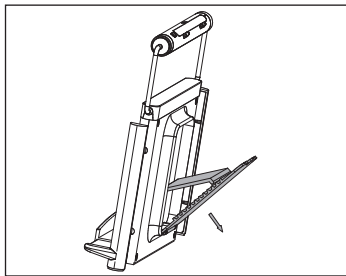
Konformitätsbestätigung / Declaration of Conformity / Déclaration de Conformité

Produkt / Product / Produit:

CD0-004 LED Leseputl Lite
 CD0-004 LED BookstandLite / Porte-livre Lite

Hersteller / Manufacturer / Fabricant:


OttLite Technologies, Inc.
 1214 West Cass Street, Tampa, FL 33606

RICHTLINIEN / DIRECTIVES:	NORMEN / NORMES:
Low Voltage Directive 2006/95/EC	EN60598-2-4 Electrical Safety for Luminaires EN 62471:2008 Photobiological Safety
EMC Directive 2004/108/EC	EN55015 EMC for Luminaires
	EN61547 EMC Immunity
	EN61000-3-2 Current Harmonics

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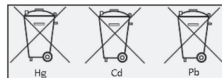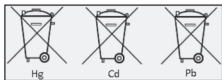
GB Battery Safety Notes

We would like to inform you that our batteries/rechargeable batteries are marketed in accordance with the current law and refer you to the following points:

1. Our products contain batteries/rechargeable batteries.
2. To prevent harmful effects to the environment and health, used batteries/rechargeable batteries must be disposed of properly at community recycling centres or returned to the merchant. Returning batteries is required by law and is completely free-of-charge for you.
3. Batteries/rechargeable batteries should only be returned in a discharged state.
4. Valuable raw materials such as zinc, iron and nickel can be

reclaimed by recycling used batteries/rechargeable batteries.

5. The batteries/rechargeable batteries can be easily removed from our devices for disposal (see User's Guide).
6. The symbol of a „crossed-out trash container“ indicates that it is illegal to dispose of used batteries with regular domestic waste.

The additional specification of the chemical sign for lead (Pb), cadmium (Cd) and mercury (Hg) indicates that these materials are contained in the

batteries/rechargeable batteries in accordance with the legally-specified limits.

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
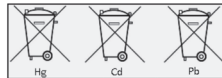
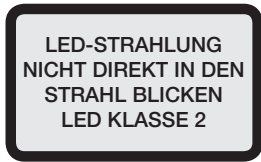
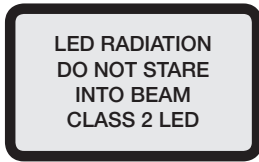
EN 55022 Class A

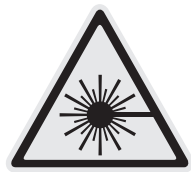EN 55024

GB And here's what else you'll find at SCHWEIZER:

- Europe's largest range of special corrective lenses
 - Special spectacle lenses for increased contrast
 - Special spectacles with magnifying effect
 - Magnifying spectacles
 - Telescopic magnifying spectacles
 - LowVision illumination and much more
- Electronic Aids
- Magnifiers for technical applications
 - Precision fold-in magnifiers
 - Precision measuring magnifiers
 - Scale measuring magnifiers
 - Thread counters
 - Wand microscopes
 - Watch makers' magnifiers
 - Work stand magnifiers, illuminated

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Copyright © Oktober 2013 A. SCHWEIZER GmbH.
Ⓒ All rights reserved.