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EN Congratulations

You have selected a SCHWEIZER quality product manufactured according to the currently valid standards and regulations. All brand products are marked “Made in Germany”. Congratulations on your purchase.

Please read this information carefully before using your product for the first time and observe the safety, cleaning and care instructions.

Intended use

This device is intended for everyday use, for reading printed matter or magnifying objects.

General information

The lens in your SCHWEIZER product is made from glass or special plastic. The optical quality of both materials is second to none. Lenses made from optical glass are ground and optically polished on both sides. The hard surface features a high chemical and mechanical resistance. Only UV-absorbing crown glass, which is also used for the production of prescription lenses, is used in the manufacturing process.

Lightweight plastic lenses are made from optically pure Plexiglas®. Computer-optimised aspheric surfaces provide clear edge-to-edge images and ensure optimal use of the entire field of view.

Cleaning and care

To achieve optimal viewing results, please clean your lens regularly and carefully with a damp, lint-free cloth (ideally a damp microfibre cloth). Never use solvents such as alcohol, benzene or other cleaning agents for cleaning lightweight plastic lenses since they can damage the lens. You may find small lines that look like hairline cracks at the edge of lightweight lenses or in the plastic frame. These are flow lines that occur during the production process; they do not affect the function or durability in any way.

Safety instructions

Fire hazard! Products containing lenses must never be left in the sun without a cover. The intensifying effect of the lens can start a fire. Risk of injury!

Never look at the sun with a lens. Do not look directly at the light source of illuminated magnifiers.

Avoid mechanical impact or shock such as banging or dropping your product, and ensure it is not exposed to moisture or to high temperatures.

There is a risk of crushing on moving parts. Ensure that the joint elements are securely fastened.

Inform other people about these risks, especially children.

Do not dispose of used batteries along with normal refuse. Please ensure proper disposal (see page 11).

EN Power supply

Some products are battery operated. Information about suitable types of battery is provided below the individual product descriptions.

The LEDs used are subject to the most stringent selection criteria. They have a lifespan of more than 50,000 hours with continuous operation.

Replacement of the LED is thus not normally required.

Symbol guide

REF

Item no.

LOT

Batch number

Manufacturer

Fragile, handle with care

Protect from moisture

Consult instructions for use

Disposal information

CE marking of conformity

Battery safety instructions


We place our batteries on the market in line with legal requirements. Please read the points below on the safe disposal of our batteries:

- To prevent harmful effects to the environment and health, used batteries must be disposed of properly at recycling centres or returned to the point of sale. Returning batteries is required by law and free of charge for you.
- Batteries should only be returned in a fully discharged state. The poles of lithium-ion batteries must be insulated with adhesive tape.
- Valuable raw materials such as zinc, iron, nickel and so on can be recovered by recycling used batteries.
- The batteries can be easily removed from our devices

for disposal (see the following pages). Magnifiers containing a built-in rechargeable battery must be disposed of completely.

- The “crossed out wheelie bin” symbol indicates that it is illegal to dispose of used batteries along with normal refuse.

The additional specification of the chemical symbols for lead (Pb), cadmium (Cd) and mercury (Hg) indicates that these materials are contained in the batteries in legally permissible quantities.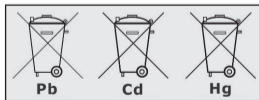


Handleuchtlupe

Tech-Line

INDUKTION

Tech-Line INDUCTION
illuminated hand magnifier

DE Linsengröße EN Lens size	Vergrößerung Magnification	Lichtfarben Light temperatures
Ø 70 mm / 22,8 mm	2 x / 4 x	4500 K
Ø 55 mm	4 x	4500 K
Ø 30 mm	8 x	4500 K
Ø 30 mm	10 x	4500 K
Ø 22,8 mm	15 x	4500 K

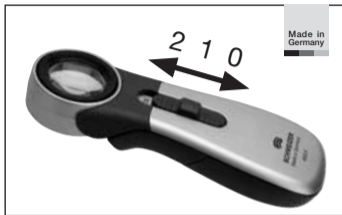DE Aus (0)/An (1)/Boost (2)

EN Off (0)/on (1)/boost (2)

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EN Connect charging pad

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EN Place magnifier on pad to charge

Ladepad zur Handleuchtlupe Tech-Line INDUKTION

Charging pad
for Tech-Line INDUCTION
illuminated hand magnifier

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EN Both indicator lights switch off when the battery is fully charged.




Handleuchtlupe

Tech-Line

CHARGING


Tech-Line CHARGING
illuminated hand magnifier

DE Linsengröße EN Lens size	Vergrößerung Magnification	Lichtfarben Light temperatures
Ø 70 mm / 22,8 mm	2 x / 4 x	4500 K
Ø 55 mm	4 x	4500 K
Ø 30 mm	8 x	4500 K
Ø 30 mm	10 x	4500 K
Ø 22,8 mm	15 x	4500 K

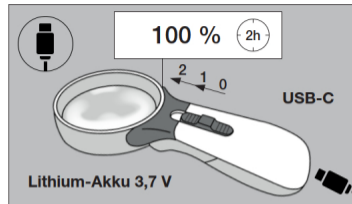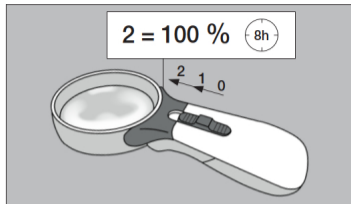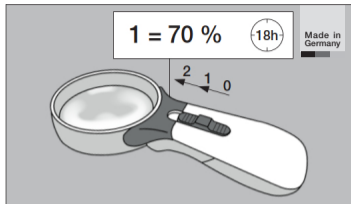DE Aus (0)/An (1)/Boost (2)

EN Off (0)/on (1)/boost (2)

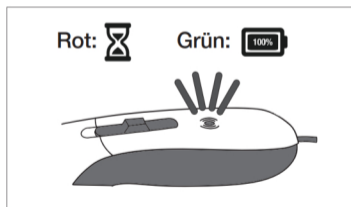DE Rot / Grün

EN Red / Green


Handleuchtlupe

Tech-Line

CLASSIC

Tech-Line CLASSIC
illuminated hand magnifier

DE Linsengröße EN Lens size	Vergrößerung Magnification	Lichtfarben Light temperatures
Ø 70 mm / 22,8 mm	2 x / 4 x	4500 K
Ø 55 mm	4 x	4500 K
Ø 30 mm	8 x	4500 K
Ø 30 mm	10 x	4500 K
Ø 22,8 mm	15 x	4500 K

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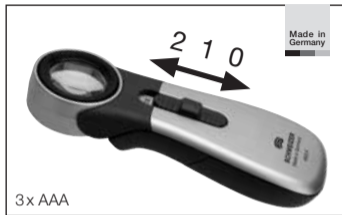EN Weak batteries will prevent the magnifier from lighting up to boost level.

Handleuchtlupe Basic-Line

Basic-Line illuminated hand magnifier

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Basic-Line STAINLESS STEEL
folding magnifier

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Basic-Line PLASTIC
folding magnifier

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Taschenlupe Basic-Line ECONOMIC

Basic-Line ECONOMIC
pocket magnifier

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Uhrmacherlupe Basic-Line GLAS

Basic-Line GLASS
watchmaker's loupe

DE Linsengröße	Vergrößerung
EN Lens size	Magnification
Ø 25 mm	3 x
Ø 25 mm	4 x
Ø 25 mm	5 x
Ø 25 mm	7 x
Ø 25 mm	10x
Ø 22,8 mm	6 x
Ø 22,8 mm	10 x

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Kopfbandlupe Tech-Line BINO LED + Wechseloptik

Tech-Line BINO LED
headband magnifier
+ interchangeable lens unit

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Binokulare Vorsetzlupe Basic-Line RIDO-CLIP

Basic-Line RIDO-CLIP
clip-on magnifier

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Basic-Line ILLUMINATED
headband magnifier

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EN With additional magnifying lens

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EN All rights reserved.
Copyright © 01/ 2026, A. SCHWEIZER GmbH
A. SCHWEIZER GmbH